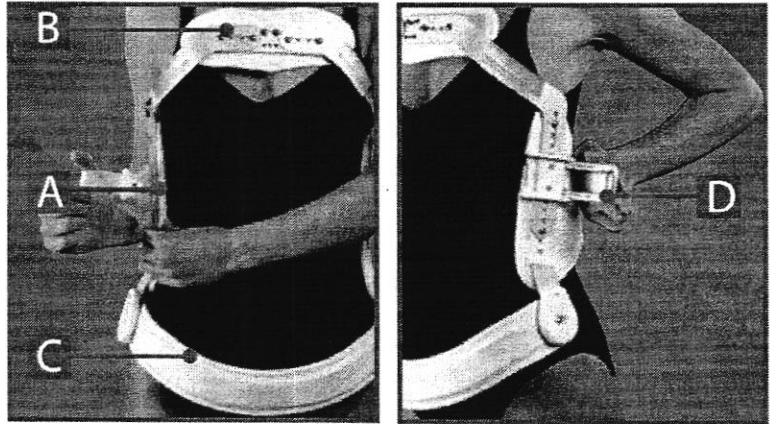


APPLICATION FOR THE PATIENT:

1. Open the closure lever and unhook the belt from the knob screw (A).
2. Position the hyperextensor on the chest, pass the belt behind the back and fasten it to the knob screw (A).
3. Position the hyperextensor so that the sternal plate is centered on the chest (B) and the pelvic band is not uncomfortable when sitting (C).
4. Fasten the hyperextensor using the closure lever (D).

**PRECAUTIONS FOR USE:**

Any pressure applied by the orthosis should not act on parts of the body with wounds, swelling or tumescence. It is advisable to wear the orthosis over a garment, avoiding direct contact with the skin. In case of doubt about the manner of application, contact an orthopedic expert. Do not wear the orthosis near flames or strong electromagnetic fields. Do not release the tension of the belt abruptly, in order to prevent sudden collapse of the trunk.

MAINTENANCE:

- Wash with a sponge dampened in warm water and neutral soap.
- Dry with a soft cloth. Check the screws periodically and tighten any loose ones.
- Contact an orthopedic expert for replacement of worn parts.
- Do not dispose of the orthosis or any of its parts carelessly.

WARNINGS:

The orthosis must be prescribed and used under medical control and applied by an orthopedic expert, who is a competent person of reference both for application and for information about safe use, in compliance with individual needs. To ensure the effectiveness, tolerability and correct function, it is necessary to apply the device with the maximum care. Any change in its structure or adjustment must be prescribed by a physician and performed by an orthopedic expert. Never alter the adjustment made by the physician or orthopedic expert. It is recommended that the orthosis be used by one patient only. In case of hypersensitivity, direct contact with the skin may cause redness or irritation. In case of pain, swelling, tumescence or any unusual reaction, contact your doctor immediately.

**Distributed by:**

DJO, LLC
Vista, CA 92018
T: 800-226-4799
T: 317-228-1144

www.Bell-Horn.com

